Quick Start Guide

FUN CUP EXPLORE KIT

Menstrual Cups

NOTICE

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ATTENTION

EN | Risk of injury! Remove piercings or jewelry in the genital area before use. Do not use the product on irritated or damaged skin. Stop using the product if you experience pain or feel unwell during use. Do not use the product if it is: 1. damaged or deformed, or 2. discolored. | Dry the cup completely before storing or using it—use a clean, lint-free towel or air dry in a clean area. | Do not simply empty and reinsert without cleaning, as this may increase the risk of infection. Empty and clean it at least twice daily. | Do not use the menstrual cup with lubricants, oils, or scented soaps. | This product is intended for single-user use only. Do not share it, even after sterilization. | Replace the cup if you notice a persistent odor. | Always wash your hands before inserting or removing the cup to reduce infection risk. | Do not clean the cup in the microwave, dishwasher, or with bleach, vinegar, alcohol, or hydrogen peroxide. Use only mild, fragrance- and oil-free soap and water. | Do not use the menstrual cup immediately after miscarriage or gynecological surgery unless advised by a healthcare professional. | Do not use this product if you are allergic to medical-grade silicone. | Suitable only for vaginal use. | Never use any foreign object to insert or remove the cup.

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FUN FACTORY Germany GmbH